

TASW CALL MANAGEMENT & PAGER CONTROL SOFTWARE

FOR TELEALARM RADIO NURSECALL SYSTEMS

TASW call management software puts you in control of your TeleAlarm nurse call system. Every activity is recorded by the computer hard drive and can be viewed or printed at will.

The main screen shows a real time log of all activities with current calls highlighted in colours of your selection. Secondary screens may be located at nurse stations or corridor areas and display only current active calls providing the benefit of hard wired system displays to a radio based system; AND, screens can be of any size, from 8 inches to 40 inches, allowing calls to be read from many metres away with up to 8 active calls displayed at any time.

The TASW software can also control alphanumeric pagers to allow staff communication anywhere within the building (or gardens!). Each pager can be set to receive selected call types for example a manager may receive EMERGENCY only, or perhaps a maintenance man may receive only fault or low battery warnings without disturbing care staff. In addition, if staff work on specific floors or wards, then their pagers may receive ONLY calls from their own area.

Date	Time	Name	Call Point
01/04/05	8:22	SLAVE ON	0 08.18.
01/04/05	8:03	SLAVE ON	0 08.23.
31/03/05	9:16	SLAVE ON	0 09.09.
31/03/05	4:23	SLAVE ON	0 08.14.
31/03/05	3:37	SLAVE ON	0 09.08.
04/11/04	15:17	EMERGENCY	21 BEDROOM 21
04/11/04	15:17	CALL	21 BEDROOM 21
04/11/04	15:16	CALL	20 BEDROOM 20
04/11/04	15:16	CANCEL	20 BEDROOM 20
04/11/04	15:16	CANCEL	20 BEDROOM 20
20/10/04	14:00	Daily Battery Check	17 Bedroom 7
20/10/04	13:53	CANCELLED	65 Fire Exit 2
20/10/04	13:53	CANCELLED	65 Fire Exit 2

Name	Call Point
EMERGENCY	BEDROOM 21
CALL	BEDROOM 21
CALL	BEDROOM 20

There is also a 'Send Message to Pager' screen to allow text messages, entered from the keyboard, to be transmitted to individual staff, groups of staff or to ALL pagers - no more missing staff or hunting for staff to transfer an important telephone call.

Date	Time	Call Type	Call Point	Position	Response Time
9/01/2007	19:20	CALL	WENTWORTH 1		0:00:07
9/01/2007	19:20	RESET	WENTWORTH 1		0:00:00
9/01/2007	19:21	CALL	WENTWORTH 2		0:00:10
9/01/2007	19:21	RESET	WENTWORTH 2		0:00:00
9/01/2007	19:22	CALL	WENTWORTH 2		0:00:58
9/01/2007	19:22	EMERGENCY	WENTWORTH 1		0:00:20
9/01/2007	19:22	RESET	WENTWORTH 1		0:00:00
9/01/2007	19:22	CALL	WENTWORTH 4	HADDON	0:00:17
9/01/2007	19:22	CALL	WENTWORTH 3		0:00:16
9/01/2007	19:22	RESET	WENTWORTH 4		0:00:00

Reports can be displayed on screen or sent to a printer to provide a hard copy at will. Standard reports allow extraction of a date and time range or a specific room details - ideal to resolve client or family complaints over response times.

Reports may also be written for specific circumstances as required.

TASW will operate on any PC running Window XP with 64 MB ram, 300 MB free disc space and 2 free serial or USB ports.

Safety Systems Distribution
Tel 0800 3282950 Fax 0800 3282951